0476

RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 25755  
 fax +39 0535 25509 e-mail: [info@rimos.com](mailto:info@rimos.com) - [www.rimos.com](http://www.rimos.com)

**GINRAM®**

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**Hysterosalpingography / sonohysterography uterine injector  
 with rigid catheter**

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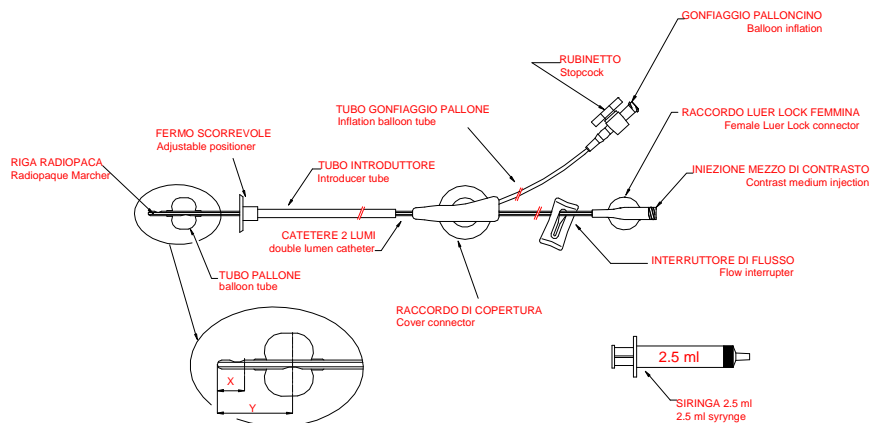
GB

### Instructions for use

Pls notice: asymmetrical inflation of the balloon does not compromise the correct working of the device as once it is placed in the uterine cavity it assumes a symmetrical shape.

1. Expose the cervix using a speculum and cleanse it with disinfectant liquid.
2. Perform a hysteroscopy to measure the depth of the uterus.
3. Take RIGIDINJECTOR out of sterile packaging and check balloon integrity (inflate and deflate with a syringe).
4. Lubricate the tip of the catheter and the balloon; insert the catheter gently into the uterine cavity through the cervical canal. Caution: the catheter must be placed 1 cm underneath detected depth. Push the catheter onward until the adjustable positioner (previously settled 1 cm underneath detected depth) is against the external os and stops.
5. Gently inflate the balloon with 2 cc air by the stopcock. **(Warning ! The Company is not responsible for any balloon bursting in case the maximum recommended inflation volume is exceeded).**  
Gently push the adjustable positioner forward against the external os to get the catheter well secured. The catheter positioned in that way has a safe and atraumatic seal in the uterus and allows to do the necessary operations.
6. Inject the contrast medium (or physiological solution in case of sonohysterography) through the second way to display the uterus and the Fallopian tubes. If necessary, interrupt the flow by closing the pipe with the clip.
7. At the end of the procedure, deflate the balloon by simply opening the tap. Withdraw the adjustable positioner with the thumb and remove the catheter.

**Caution:** to be used only after performing a hysteroscopy to avoid perforation of the uterine fundus. Read the instruction before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

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