0476

RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 – 25755 –
fax +39 0535 – 25509 e-mail: info@rimos.com – http://www.rimos.com

FERTIRAM®

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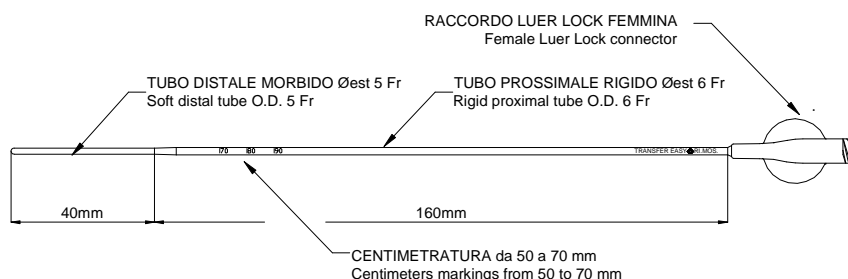
GB Instructions for use

Introduce the speculum, expose the cervix and if possible, gently remove the cervical mucus. Lightly swab the uterine cervix with a gauze soaked in embryo culture medium. Connect the catheter to the insuline syringe and introduce the catheter previously loaded with embryos into the uterine cavity. Release the embryos in the preselected point without touching the fundus. Graduation marks on the catheter aid correct placement of the embryos and echogenic tip allows ultrasound-guided embryo transfer. Once embryos have been unloaded, slowly withdraw the catheter and check it under stereomicroscope for any retained embryo.

Caution: Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose after use according to the laws in force. For use only under trained medical staff.

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FERTIRAM products are manufactured from biocompatible materials suitable for human assisted fertilization, MEA & LAL tested and Beta Sterilized.