0476

RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 – 25755 –
fax +39 0535 – 25509 e-mail: info@rimos.com – http://www.rimos.com

FERTIRAM®

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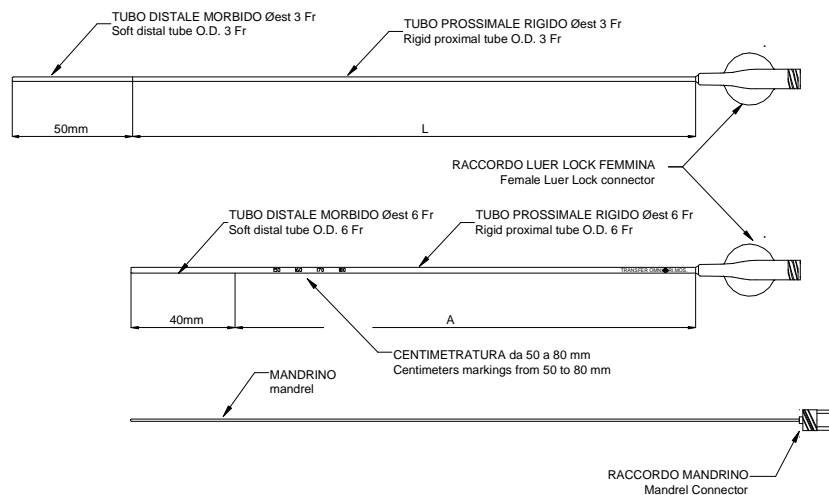
GB

Instructions for use

Introduce the speculum, expose the cervix and if possible, gently remove the cervical mucus. Lightly swab the uterine cervix with a gauze soaked in embryo culture medium. Introduce the guide-catheter (with mandrel inside) through the cervix just beyond the internal OS. Remove the mandrel from guide-catheter. Gently advance the transfer catheter (previously connected to the insuline syringe and loaded with embryos) through the guide-catheter inside the uterine cavity until reaching the pre-selected point. Do not touch the fundus to avoid any stimulation of uterus contractility. The centimeter markings placed on the guide catheter are helpful to determine the proper operative depth; those on the transfer catheter help to get tips alignment. Echogenic tip allows ultrasound visualization. Once the embryo-transfer has been completed, remove both catheters simultaneously to avoid possible aspiration episodes and check under stereomicroscope for any retained embryo.

Caution: Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose after use according to the laws in force. For use only under trained medical staff.

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FERTIRAM products are manufactured from biocompatible materials suitable for human assisted fertilization, MEA & LAL tested and Beta Sterilized.

0476

RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 – 25755 –
fax +39 0535 – 25509 e-mail: info@rimos.com – <http://www.rimos.com>