RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 – 25755 –
fax +39 0535 – 25509 e-mail: info@rimos.com – http://www.rimos.com

FERTIRAM ®

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GB

The catheter is stored at a temperature of 37° C in sterile conditions.
Catheter washing is not required.

Patients with anteverted uterus should fill urinary bladder in order to reduce uterine angle.
An ultrasound hysteroscopy and measurement of cervical length are suggested for a correct catheter positioning.

SUGGESTED PROCEDURE

Introduce the speculum, expose the cervix and gently remove the cervical mucus. Gently “touch” the uterine cervix with a gauze soaked with embryo culture medium.

- Place the blue stopper on the introducer on the cervical length bases. Insert the introducer in the cervical canal until the blue stopper gets in contact with uterine cervix. In this way, the catheter tip is able to reach the internal uterine ostium (IUO), without overcoming it.
- At the same time, in the laboratory, aspirate in the catheter as following: approx 1 cm of air, then 0.5-1 cm of culture medium with the embryos, followed by 1 cm of air.
- Gently take out the yellow mandrel and introduce the catheter previously loaded with the embryos. When first strongest mark on the catheter is in correspondence of introducer entry, catheter tip matches introducer tip. At this point, push the catheter and bring it forward over IUO into uterine cavity. Making use of the graduation marking placed on the catheter, the preselected transfer point can be reached, on the basis of the previous hysteroscopy.
- Keep then the catheter still and push the plunger forward to release the embryos into the uterine cavity. The catheter echogenicity allows ultrasound-guided embryo transfer.
- Once embryos have been released, gently withdraw both catheter and introducer, checking possible presence of any retained embryos.

Caution: read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose after use according to the laws in force. For use only under trained medical staff.

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